

USE ONLY
THE POWER
SUPPLIED
DEVICE
WITH THIS
PRODUCT

OPERATE
YOUR
SYSTEM
ACCORDING
TO THIS
MANUAL

WARNING!

To reduce risk of burns, fire, electric shock, or injury to persons.

1. Your *MED-LIFT* product should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
2. Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer. Never operate the *MED-LIFT* product if it has a damaged cord or plug, or if it is not working properly. If it has been dropped or damaged, or dropped into water. Contact your dealer for instructions on returning it for examination and repair.
3. Close supervision is necessary when this appliance is used by, on, or near children or disabled persons.
4. Do not use outdoors.
5. Keep the cord away from heated surfaces.
6. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
7. If you are subject to any medical condition involving the use of medical device products, and/or if you are under a health professional or physician's care, please contact them before using this product.
8. If there is a malfunction of any kind, contact your dealer for instructions on returning it for examination and repair.

MAINTENANCE INSTRUCTIONS:

No user maintenance of this product is required. If the System does not function, be sure...

- (a) The TRANSFORMER or ADAPTOR is plugged into an electric outlet.
- (b) The TRANSFORMER or ADAPTOR cord is plugged into, and fully engaged in its receptacle.

If there is a malfunction of any kind, contact your dealer for instructions on returning it for examination and repair.

If you need assistance for any reason, please contact us:
Write to: Customer Service InSeat Solutions, Inc. Corporate Office
11839 Smith Avenue, Santa Fe Springs, CA 90670

call or fax us at:

1-562-447-1780 ext. 204
FAX: 1-562-447-1798

www.inseat.com

DANGER!

To reduce risk of electric shock.

1. Always unplug the product from the electrical outlet immediately after using and before cleaning.
2. Do not reach for a product that has fallen into water. Unplug immediately.
3. Do not use while bathing or in a shower.

Read all instructions before using the *MED-LIFT* System.

IMPORTANT SAFETY INSTRUCTIONS



MED-LIFT

**Massage, LIFT
and HEAT System**

OWNER S MANUAL and OPERATING INSTRUCTIONS

Welcome to the world of Massage, Lift, and Heat!

You are now the proud owner of another great *MED-LIFT* product.

Your new *MED-LIFT* massage, lift, and heat system brings the comforting world of massage, lift, and heat technology and places it at your fingertips. Now you can enjoy the relaxing feel of vibrating massage or the soothing comfort of heat on your tired, knotted muscles.

We know you will enjoy your new *MED-LIFT* massage, lift, and heat system product.

Limited Manufacturer's Warranty

Patents Pending

©Copyright 2001 InSeat Solutions, Inc. Santa Fe Springs, CA

OMLU1PH

0608

#11690U-30

OPERATING INSTRUCTIONS

POWER

Activates Massage System

- Turn On: Press PWR.
- Turn Off: Press PWR again.

30 minute Timer:

To restart after Timer shuts off:

- Press PWR or MSG key.

MASSAGE

(Green light indicates activation)

- Actual placement of massage motor may vary with each manufacturer and chair type.

LIFT (DOWN ▾)

Activates actuator motor

- Actual placement of actuator motor may vary with each manufacturer and chair type.



HEAT

Activates Heat Element.

- Turn On: Press HEAT
- Turn Off: Press HEAT.

Works with massage or independently.

30 minute Timer

To continue after Timer shuts off, press HEAT button again.

LIFT (UP ▲)

Activates actuator motor

- Actual placement of actuator motor may vary with each manufacturer and chair type.

NOTE: Constant wand lights indicates that there is active power in the wand and will dim down after 10 seconds of non-use.

SAVE THESE INSTRUCTIONS!